KALT-/WARM-MEHRFACHKOMPRESSE **HEAT/COLD MULTIPLE-USE COMPRESS** **COMPRESSE MULTIPLE FROID/CHAUD**

DE AT CH

KALT-/WARM-MEHRFACHKOMPRESSE

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GB IE

HEAT/COLD MULTIPLE-USE COMPRESS

Instructions for use

FR BE

COMPRESSE MULTIPLE FROID/CHAUD

Mode d'emploi

NL BE

KOUDE-/WARMTE-MEERVOUDIG KOMPRES

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CZ

STUDENÁ/TEPLÁ MULTIKOMPRESSE

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ES

COMPRESAS MÚLTIPLES FRÍO/CALOR

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Herstellernummer

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Hersteller

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Congratulations on your purchase of your new product.

You have thereby opted in favour of a high quality product. The use instructions are a constituent part of this product. They entail important information with regard to the safety and use. Familiarise yourself before using the product with all use and safety instructions. You should only use the product in perfect condition, as described and for the indicated application purposes. Please keep the packaging, including this leaflet for the shelf life of your bandage and they should also be passed along, in case you pass the product on to third parties.

This product is compliant with Directive 93/42/EEC concerning medical devices.

Scope of delivery

- 1 fastening strap
- 1 heat/cold multiple-use compress
- 1 use instructions

Material: 100 % Polyester

Intended Use

Only for private use! The product is suitable for the microwave oven.

Indications: heat application:

- Check whether a heat treatment alleviates your muscle pain.
- You should loosen your muscle tension during a heat treatment.

Indications: cold application:

Mitigates bruises, sprains and swellings through cooling.

Do not use:

- in the case of limited faculty of perception of cold and heat
- in the case of arterial circulatory disorders and Raynaud's syndrome
- during physical strain and sparring exertion
- in the case of open injuries or skin diseases in the treatment area
- during the period of the intake of analgesic agents or in the case of temperature sensitivity
- heat application in the case of fever or inflammations

Possible side effects

- You should immediately take off the fastening strap, if symptoms such as tingling, numbness, pain, swellings and skin discolorations manifest themselves.
- Skin reactions and irritations, could be triggered.
- Do not exceed, the specified wearing periods.
- Please consult a medical physician immediately, insofar as you experience any health problems or discomfort!

Heat application

Heating in a microwave oven

Set the microwave oven to 600 watts and heat the compress for approx. 15–25 seconds. Check whether a temperature, which is convenient for you, has been reached. You can optionally heat for an additional 10 seconds.

Heating in a water bath

Boil water in a saucepan. Take the saucepan off the cooker and place the compress for 1–3 minutes in the water. Check whether a temperature, which is convenient for you, has been reached.

Apply the compress for approx. 20–40 minutes.

Cold application

Place the heat/cold multiple-use compress in the refrigerator, to cool for one hour at 5 °C. The compress is not suitable for the freezer!

Application time: approx. 10 minutes.

Safety instructions

Risk of injury!

- Keep out of the reach of children and only use under adult supervision.
- Comply with the specified pre-cooling and heating times.
- The compress and the fastening strap must be checked before each use, for cracks or other damage.
- Only use the product, insofar as it is in a perfect condition.
- The cooled or heated compress must never be placed directly on the skin. Use the fastening strap or wrap the compress in a cloth before use.
- Insofar as you experience problems while wearing, remove the fastening strap immediately and you should moreover also consult a medical physician, in case the health problems persist.
- Do not over tighten the fastening strap, since this could impair your blood circulation.
- The compress may not be used while sleeping.
- The skin must be cleaned thoroughly, prior to the use of the compress. The skin must be free of gels, fats, ointments, oils or similar substances.

Cleaning and care

Fastening strap:

The fastening strap should be washed, prior to the first use. You should for this purpose close the hook and loop fastener. The product should be washed again, after the third time of use.

Cold/heat-multiple-use compress:

The compress should be cleaned after each use with a damp cloth.

Storage:

Store the product dry and warm.

Disposal instructions

Please dispose of the product in an environmentally appropriate manner through an approved disposal center or your community waste facility. You should thereby observe the currently applicable regulations.

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Horizonte Textil GmbH, Am Sandtorkai 23, D-20457 Hamburg